

## One Way Seating Aid

### Description:

The One Way Seating Aid enables a patient to be moved to the back of a seat or wheelchair with ease, yet prevents the patient sliding forward whilst seated.

The textured, rubberized non-slip fabric grips easily to most seating surfaces, whilst two strips of raised pile inside the Seating Aid allow effortless slide in one direction and prevent movement in the opposite direction.

Hand or machine wash up to 60°C.

### Dimensions:

Width: 45cm

Length: 43cm

### Instructions for use:

Using the arrow on the label as a guide, place the One Way Seating Aid on the front of the chair positioning the direction of slip toward the back of the chair. By allowing one third of the Seating Aid to hang over the front of the chair, the patient is able to position their bottom on the chair and slide easily to the back of the chair. The Seating Aid will then prevent any forward slip.



Check the patient's seating position regularly, particularly if they are at risk of pressure complications.

*The aid is a Class 1 Medical Device (non-invasive and non-sterile).*