

Bed Ladder Strap

Description:

The Bed Ladder Strap is a strong band of webbing with four equidistant handholds along its length. The Bed Ladder Strap is primarily used to assist patients with independent sitting.

All loops are attached to the main webbing with rectangular and diagonal stitching. All ends are sewn under to present a smooth, comfortable handling surface.

The Bed Ladder Strap is machine washable to 80°C.

Dimensions:

L 186cm, W 3.5cm, Loop Circumference 28cm

Weight Capacity – 250kg SWL

Instructions for use:

Attach the Bed Ladder Strap to the end of the patient's bed by looping the strap around the bed frame, and laying the strap along the length of the bed, in reach of the patient. The hand holds are used by the patient to assist in transfer from supine to sitting.



The aid is a Class 1 Medical Device (non-invasive and non-sterile)