

Chair Belt

Description:

The chair belt is an adjustable restraint belt designed to fasten behind the chair legs of a seated patient. The belt will adjust to fit a lounge chair, wheelchair or commode. The Chair Belt is machine washable to 80°C.

Dimensions:

Length: 2.5m extended.



Instructions for use:

The belt is placed around the pelvic area, below the waist, depending on the individual's seating position. The two side straps should be wound behind the chair frame at approximately the same level as the patient's buttocks.

Safety belts must be fitted correctly and the patient kept under close supervision. Do not buckle behind the patient's back.

WARNING: A patient restrained by a chair belt must be supervised and checked-on every 15 minutes. Please see the relevant guidelines from your State Health Regulators.



The aid is a Class 1 Medical Device (non-invasive and non-sterile)