

## Leg Lifter

### Description:

The Leg Lifter is designed to aid patients lifting their leg on or off the bed independently.

The Leg Lifter is made from strong webbing with a plastic insert to keep the lifter semi-rigid. This enables the patient to place the loop over their foot without having to bend forward to touch their toes.

Hand or machine wash up to 80°C.

### Dimensions:

Length: 90cm

### Instructions for use:

To lift the leg into position, the user needs to be sitting on the side of the bed. Place the foot loop over one foot. Lean back to create a comfortable lift by using body weight. Users who have difficulty using their fingers can insert their arm through the hand-holds for extra support.

To lift both legs together, cross one foot over the other and place the lower foot into the leg lifter loop.



*The aid is a Class 1 Medical Device (non-invasive and non-sterile).*