

User Manual

Ellipse Lite Rollator



Aluminum Rollator

Thank you for purchasing the Peak-Care Ellipse Lite Rollator.

The Walking aid is suitable for a single user with limited mobility. The Ellipse Lite is ideal for indoor and outdoor use.

Kindly take time to go through this manual which gives you more details about your Ellipse Lite Rollator. Kindly keep this booklet handy and use it as and when required.

Ellipse Lite is warranted to be free of defects in material and workmanship for twelve months from the date of purchase

This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit.

This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries, please contact the distributor

Kindly attach your receipt to this booklet

PARTS LIST



- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing hand grip | 8. Bag |
| 3. Rear reflector handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Rest | |

INTRODUCTION

The Walking aid is suitable for a single user with limited mobility. The Ellipse Lite is ideal for indoor and outdoor use.

ELLIPSE LITE PACK INCLUDES

<p>1 Ellipse Lite Rollator</p> 	<p>2 Bag</p> 	<p>3 Back Rest</p> <p>1027704</p> 	<p>4 User Manual</p> 
---	---	---	---

UNFOLDING ELLIPSE LITE

Push downwards on the seat rails to engage the locking mechanism.




HEIGHT ADJUSTMENT

Press the red release button located below the handgrips as shown and at the same time either push the handle down (lowering handle height) or pull the handle up (raising handle height), releasing the push button will automatically lock the handle at desired height.



ATTACHING THE CARRY BAG

Fold Ellipse Lite, Remove the stopper from the tube on either side of the seat, push the sliding block from bag into the guiding sleeve and put the stopper back in place.

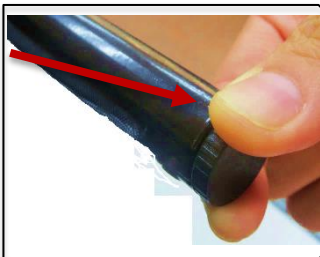

BRAKING / LOCKING

Pull both brake levers upwards to apply the brakes while the Ellipse Lite is moving. Always brake slowly and steadily

Press both brake levers down to apply locks while the Ellipse Lite is stationary. Pull up the brake levers and release the locks



The effect of the brakes can be affected by abrasion of the wheels. Test the brakes

ADJUSTMENT OF BRAKES

To adjust the braking strength, adjust the brake by pressing the screw housing downwards and then left or right



SITTING

Position the Ellipse Lite on an even and solid surface.

Lock the brakes (parking position). The Ellipse Lite must stand and must not slip.

Sit straight up and do not lean backwards





FOLDING/ TRANSPORT/ STORAGE

To fold, pull seat upwards by the handle of the seat plate



Once the Ellipse Lite is folded, the red Locking clip will secure and stop the walker from opening whilst lifting in or out of a vehicle. See below:

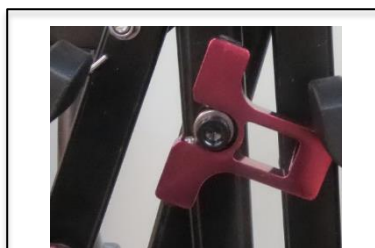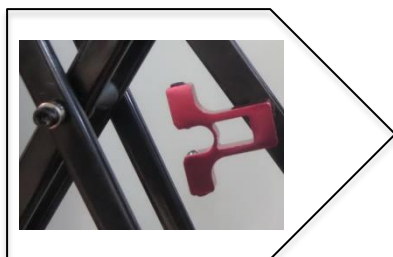

CLEANING AND MAINTENANCE

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

SAFETY INSTRUCTIONS

IMPORTANT • Do not propel yourself or have anyone push you whilst you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

- Use only as walking aid
- Use only on even and solid surfaces
- Maximum user weight: 136 Kg
- Regularly check the screws and fasteners on the Ellipse Lite and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 5 Kg
- Do not expose the product to temperatures above 40 °C for an extended period of time, as this can cause the Ellipse Lite parts to overheat and burn the user
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Ensure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace castors when tyres are significantly worn or damaged.

SPECIFICATIONS

	<i>Small</i>	<i>Medium</i>	<i>Large</i>
<i>Overall length</i>	595mm	595mm	595mm
<i>Overall width</i>	490mm	490mm	490mm
<i>Handle (Highest)</i>	945mm	978mm	1055mm
<i>Handle (Lowest)</i>	821mm	840mm	931mm
<i>Seat depth</i>	220mm	220mm	220mm
<i>Seat width</i>	454mm	454mm	454mm
<i>Seat height</i>	490mm	523mm	595mm
<i>Product weight</i>	8Kg	8.2Kg	8.7Kg
<i>Weight limit</i>	136kg	136kg	136kg
<i>Wheel diameter Rear</i>	203mm	203mm	203mm
<i>Wheel diameter Front</i>	242mm	242mm	242mm



Level 1, 149 Princes Highway
Unanderra NSW 2526
Australia
Tel: +61 2 4272 2688. Fax: +61 2 4272 2699
www.peak-care.com.au

Distributor:



- Page intentionally left blank -

- Page intentionally left blank -