

User Manual

Ellipse Super Lite Rollator



Carbon Fibre Rollator

Thank you for purchasing the Peak-Care Ellipse Super Lite Rollator.

The Walking aid is suitable for a single user with limited mobility. The Ellipse Super Lite is ideal for indoor and outdoor use.

Kindly take time to go through this manual which gives you more details about your Ellipse Super Lite Rollator. Kindly keep this booklet handy and use it as and when required.

Ellipse Super Lite is warrantied to be free of defects in material and workmanship for twelve months from the date of purchase.

This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit.

This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries, please contact the distributor.

Kindly attach your receipt to this booklet

List of Parts




- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing hand grip | 8. Bag |
| 3. Rear reflector handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Rest | 12. Safety bow |

This pack includes




UNFOLDING ELLIPSE SUPER LITE

Position Ellipse on its safety bow and lift it

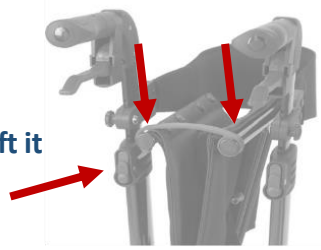

Push downwards on the seat rails to engage the locking mechanism.

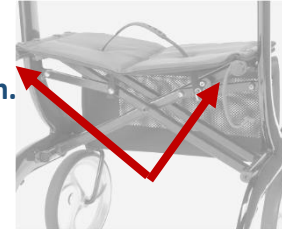



HEIGHT ADJUSTMENT

Press the red release button located below the handgrips as shown and at the same time either push the handle down (lowering handle height) or pull the handle up (raising handle height), releasing the push button will automatically lock the handle at desired height.



ATTACHING THE CARRY BAG

Fold Ellipse Super Lite, Remove the stopper from the tube on either side of the seat, push the sliding block from bag into the guiding sleeve and put the stopper back in place

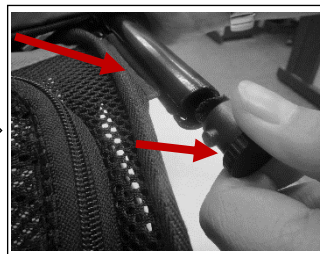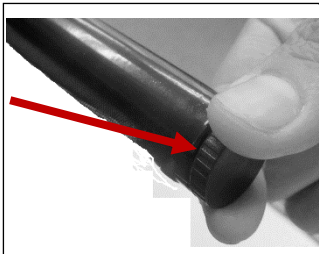

BRAKING / LOCKING

Pull both brake levers upwards to apply the brakes while the Ellipse Super Lite is moving. Always brake slowly and steadily





Press the end of the brake lever as shown to apply the hand brake. Pull up the end of the lever to release the hand brake.



The effect of the brakes can be affected by abrasion of the wheels. Test the brakes regularly.

ADJUSTMENT OF BRAKES

To adjust the braking strength, adjust the brake by pressing the screw housing downwards and then left or right

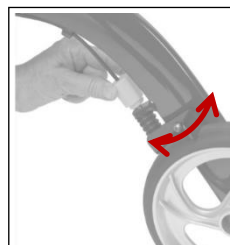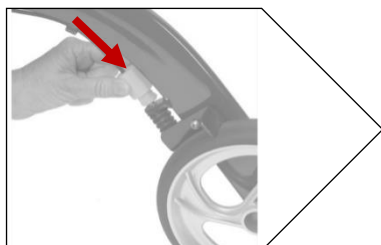

SITTING

Position the Ellipse Super Lite on an even and solid surface. Lock the brakes (parking position). The Ellipse Super Lite must stand and must not slip. Sit straight up and do not lean backwards




FOLDING/ TRANSPORT/ STORAGE

To fold, pull seat upwards by the handle of the seat plate



Once the Ellipse Super Lite is folded, use the red safety clip to secure the seat. This will stop the walker from opening whilst lifting in or out of a vehicle. See below:

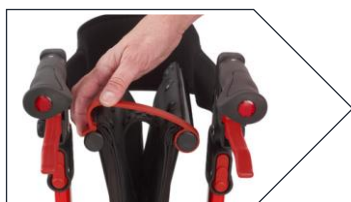

SAFETY INSTRUCTIONS

IMPORTANT → Do not propel yourself or have anyone push you whilst you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

- Use only as walking aid
- Use only on even and solid surfaces
- Maximum user weight: 120 Kg
- Regularly check the screws and fasteners on the Ellipse Super Lite and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 5 Kg
- Do not expose the product to temperatures above 40 °C for an extended period of time, as this can cause the Ellipse Super Lite parts to overheat and burn the user
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Ensure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an inclined or uneven ground.
- Replace castors when tyres are significantly worn or damaged.

CLEANING AND MAINTENANCE

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

SPECIFICATIONS

	Small	Medium	Large
Overall Length (mm)	660	680	706
Overall width (mm)	600	600	600
Handle (highest) (mm)	850	960	1050
Handle (lowest) (mm)	760	835	920
Seat depth (mm)	220	220	220
Seat Width (mm)	450	454	454
Seat height (mm)	480	520	600
Product nett weight (approx.) (kg)	6.26	6.63	6.69
Weight Limit (kg)	120	120	120
Wheel diameter Rear (mm)	203	203	203
Wheel diameter Front (mm)	203	242	242



Level 1, 149 Princes Highway
Unanderra NSW 2526
Australia
Tel: +61 2 4272 2688. Fax: +61 2 4272 2699
www.peak-care.com.au

Distributor:

- Page intentionally left blank