

Shower Dignity Drape

Description:

The Shower Dignity Drape protects patient's modesty to and from the shower room. An adjustable neck-line ensures patient comfort.

The Shower Dignity Drape is made from shower proof Nylon fabric and will not absorb water.

Hand or machine wash up to 80°C. Do not tumble dry – air dry only.

Dimensions:

Length: 100cm

Neck circumference: Adjustable

Instructions for use:

Slip the drape over the patient's head and fasten the collar around the patient's neck. Air dry after use.

The aid is a Class 1 Medical Device (non-invasive and non-sterile).

