

Stand Up Alarm: Nurse Call

Description:

The Stand Up Alarm is a good early warning system to monitor patients whose unsupervised wandering will present a danger to themselves. The slip-resistant mat presents no danger to the resident, but provides nurses with an early warning signal.

The alarm plugs directly into existing nurse call systems via the quarter-inch mono-plug. No power plug is required. If your Nurse Call system works with different plug attachments, we will do our best to find a solution for your facility.

Dimensions:

Mat measures: 120cm (L) x 61cm (W)

Cord length: 240cm

Instructions for Use:

Place the alarm on the floor beside the resident's bed. The alarm is best placed on the side most regularly used to exit the bed – either the patient's strongest / most comfortable side or the side closest to the bathroom and door.

Check that the edge of the non-slip mat is under the bed. This will eliminate the risk of tripping on the edge of the mat and will ensure good contact when the resident steps onto the mat. Ideally, they will stand close to the centre of the mat upon standing.

Plug the cord into the Nurse Call system and double check the mat is working by standing on the mat.

The Stand Up Alarm should be regularly cleaned in line with your regular floor cleaning protocol.

Please contact us if your Nurse Call system works with a different plug attachment – we will do our best to find a solution for you.