

Ellipse Outdoor Rollator

Product Code: 9346376000550

Specifications

<i>Overall length</i>	495mm
<i>Overall width</i>	525mm
<i>Handle (Highest)</i>	1015mm
<i>Handle (Lowest)</i>	895mm
<i>Seat depth</i>	260mm
<i>Seat width</i>	525mm
<i>Seat height</i>	560mm
<i>Product weight</i>	10.5Kg
<i>Weight limit</i>	136kg
<i>Wheel diameter Rear</i>	280mm
<i>Wheel diameter Front</i>	280mm



Peak Care Equipment
149 Princes Highway
Unanderra NSW 2526
Tel: +612 4272 2688
www.peak-care.com.au

Distributor:

User Manual

Thank You!

Thank you for purchasing the Peak-Care Ellipse Lite Outdoor Rollator.

The Walking aid is suitable for a single user with limited mobility. The Peak-Care Ellipse Lite Outdoor Rollator is ideal for outdoor use. Kindly take time to go through this manual which gives you more details about your Ellipse Super Lite Rollator. Kindly keep this booklet handy and use it as and when required.

Peak-Care Ellipse Lite Outdoor Rollator is warrantied to be free of defects in material and workmanship for twelve months from the date of purchase.

This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit. This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries, please contact the distributor.

8. CLEANING AND MAINTENANCE


Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

9. SAFETY INSTRUCTIONS

- Do not propel yourself or have anyone push you whilst you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.
- Use only as walking aid
- Use only on even and solid surfaces
- Maximum user weight: 136 Kg
- Regularly check the screws and fasteners on the Ellipse Lite Outdoor and re-tighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 5 Kg
- Do not expose the product to temperatures above 40 °C for an extended period of time, as this can cause the Ellipse Lite Outdoor parts to overheat and burn the user
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Ensure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace castors when tyres are significantly worn or damaged.

5. ADJUSTMENT OF BRAKES

To adjust the braking strength, adjust the brake by pressing the screw housing downwards and then left or right

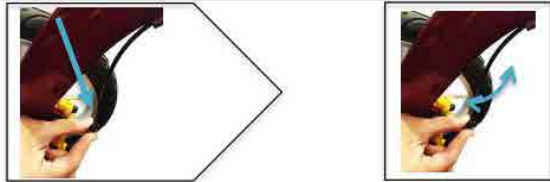


6. SITTING

Position the Ellipse Lite Outdoor on an even and solid surface. Lock the brakes (parking position). The Ellipse Lite must stand and must not slip. Sit straight up and do not lean backwards.



7. FOLDING/ TRANSPORT/ STORAGE

To fold, pull seat upwards by the handle of the seat plate. Once the Ellipse Lite Outdoor is folded, use the red safety clip to secure the seat. This will stop the walker from opening whilst lifting in or out of a vehicle.

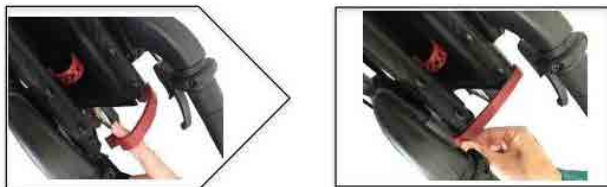

PARTS LIST



- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing hand grip | 8. Bag |
| 3. Rear reflector handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Rest | 12. Safety bow |

INTRODUCTION

The Walking aid is suitable for a single user with limited mobility. The Ellipse Lite Outdoor is ideal for outdoor use.

ELLIPSE LITE OUTDOOR PACK INCLUDES



1. UNFOLDING ELLIPSE LITE OUTDOOR

Position Ellipse on its safety bow and lift it up

Push downwards on the seat rails to engage the locking mechanism.



2. HEIGHT ADJUSTMENT

Press the red release button located below the handgrips as shown and at the same time either push the handle down (lowering handle height) or pull the handle up (raising handle height), releasing the push button will automatically lock the handle at desired height.



3. ATTACHING THE CARRY BAG

Fold Ellipse Lite Outdoor, Remove the stopper from the tube on either side of the seat, push the sliding block from bag into the guiding sleeve and put the stopper back in place.



4. Braking/ Locking

Hold the lever to brake



Parking brake: using the finger to hold the small switch and pull the lever; it can be parked at the third position.



Free parking brake: holding the brake lever and the small switch will release automatically; in the meantime, to release the parking brake for using rollator freely.

